MultiEquipment. Ref F338A

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Cavitación
Cavitation
Cavitation
Kavitation
Cavitazione

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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General information

Please check if the article is perfectly packed and intact when you receive it. Otherwise, please contact your distributor or the manufacturer. Package should include:

- 1 main unit
- 1 power cord
- 1 cavitation electrode
- 1 foot controller

Cavitation is a non-invasive treatment based on high-power ultrasound (with a 40 KHz frequency). By the high-speed mechanical vibration of ultrasonic, empty microbubbles are generated in the liquid situated in the interior of the adipose tissue. This repeated action causes the accumulation of these microbubbles to implode and collapse. The result is the destruction of adipose cells transforming the fat nodules into liquid substances that are removed by the lymph in a natural way.

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Operation method

1. Connect the power cord to the rear of the equipment. Then also the foot controller and the cavitation electrode.
2. Press the power switch ON.
3. Apply enough gel on the area to be treated.
4. Press +/- button of ENERGY and set the minimum intensity. Adjust the time up to 5-8 min. by pressing +/- button of TIMER. Then press the START/PAUSE button to begin the treatment.
5. Step on the foot controller to start working.
6. Choose the proper intensity according to the patient needs and feelings.
7. Once the treatment has finished, release the foot controller and press the START/PAUSE button. Then turn the machine off and sterilise all the used accessories.

3. Remove the fat nodules in liquid substances from the upper stomach up to the axillary lymph node.
4. Remove the fat nodules in liquid substances from the lower stomach up to the inguinal lymph node.
5. Waist: remove the fat nodules in liquid substances from the waist up to the inguinal lymph node

HIP

Keep moving the electrode forward and backward like an 8, removing like that the fat nodules in liquid substances up to inguinal lymph nodes.

Recommendations

LEGS

1. Thigh: Keep moving the electrode as following the directions given:
Inner thigh → anterior thigh → external thigh, then up to the inguinal lymph node.
When treating the inner thigh and external thigh, it is recommended to move the electrode forth and back like a Z or an 8. For the posterior thigh, move the electrode from the posterior area up to the inguinal lymph node.
2. Calf: Move the electrode like a Z, forth and back along the calf to remove the fat nodules through the lymph node.

BACK AND ARM

1. Move the electrode from the upper central part of spine to the axillary lymph node.
2. Move the electrode from the posterior area of the arm up to the axillary lymph node.
3. Move the electrode from the anterior are of the arm up to the axillary lymph node.

ABDOMEN AND WAIST

1. Keep moving the electrode from the central lower part of vertebrae through the frontal area of the waist up to the inguinal lymph node. (Attention: Do not apply on the bulge bones.)
2. Keep moving the electrode in clockwise direction from descending colon → transverse colon → descending colon → groin. See the photo below.

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Remarks

- The surface of the electrode should be totally in contact with the treated area of the skin.
- Once you release the foot controller, the electrode will stop working.
- When you finish, clean the treated area of the skin with a hot towel.
- Apply the electrode to the patient's skin by making constant circles or by moving straight without interrupting the contact between the skin and the electrode. Never keep the electrode without on the same area during a long period of time as the patient could be burned.
- This instrument is conceived for abdomen, thighs and buttocks. The face and the areas where is very close to the bones, such as joints, should be avoided in this treatment due to the strong ultrasonic frequency.
- Keep the equipment away from humid spaces and inflammable things.
- To prevent any electronic interference, please do not place any high- frequency instrument around cavitation during operation. If cavitation is interfered during operation, the instrument will suddenly shut down or the display screen will flash. If so, please power the instrument off and restart it.
- Prevent the instrument from falling down and colliding with others.
- Maintain the main unit and accessories periodically. Pay attention to prevent water from coming into.
- The instrument should be covered with a bag to avoid moisture and dust damaging the equipment if it is not used for a long time.
- Please turn the equipment on for one hour every 15 days. This is to prevent shortening the lifetime of the electronic components due to moisture.

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Maintenance

1. Handle the accessories with care, clean it after each use and keep them properly at the end. To avoid any damage or abnormality, hold the handle for the electrode, instead of for the connecting wire.
2. Clean the accessories with a cloth humid in water with some drops of detergent. A saline solution or a neutral detergent are recommended for cleaning the surface of the electrode.
3. Foresee a 30-minute session each time. Before having a second treatment, turn the machine off for 5 min. to cool the electrode. To avoid any damage of the electrode, pay attention not to turn the cavitation function on without applying the electrode on skin.

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Secondary effects and contraindications

The cavitation treatment is contraindicated in:

1. Serious heart diseases, gastric ulcer and other serious stomach diseases.
2. Duodenal ulcer.
3. Diabetes, hypertension.
4. Pregnancy and lactation.
5. Menstrual.
6. Recent abdominal operation in less than 3 months.

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TECHNICAL SHEET

Weights and sizes	Packing size (cm)	33x27,5x10
	Gross weight (kg)	4,2
	Net weight (kg)	3,1
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Working frequency (KHz)	36KHz - 40,2 KHz
	Working power (W)	35

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